

SureMed by Omnicell weekly pill packs


PRODUCT IMPROVEMENT

The template used in this product has been improved and has changed from foam to plastic.

- The new template will enhance the card sealing process.
- It is made of recyclable plastic and is able to be recycled after use, making it more environmentally friendly.
- The new template has been ergonomically designed with grip holders on either side of the template. The new template should be easier to use for both right handed and left handed pharmacists.


If you have any questions, please contact + 44 (0)161 413 5333.

Assembling the packs

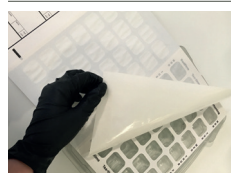

1: Peel

Remove the pressure sensitive liner from right to left to reveal the sealing surface. Put the card on the filling template (glue side up). The rest of the liner **must** be left attached to the foil side until you've completed dispensing.

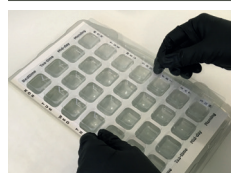


2: Insert

Once the card is on the filling template, immediately insert the blister sheet into the card starting with the top row. Lower the rest of the sheet into the remaining holes. Apply the roller to ensure the blister sheet is securely held by the adhesive to the card.



3: Dispensing

Using the reverse side as a guide, fill the appropriate blister with the correct medication.

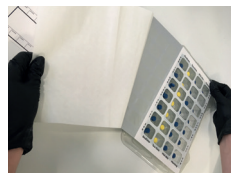

4: Remove Liner

Only when you are ready to seal the pack remove the pressure sensitive liner.



5: Fold

As soon as the liner is removed, fold the sealing surface to cover the filled blister.



6: Seal

Using the roller apply firm pressure to the whole surface to seal. Place the correct patient medication labels on the instruction flap. Indicate the shape, colour etc. of medication under each label that is applied. The SureMed by Omnicell pack is now complete.



DOs

Do - apply firm pressure using the roller when sealing the SureMed by Omnicell packs.

Do - store the cards and blisters in their original boxes.

Do - store weekly pill packs and blisters in a dry, indoor and air-conditioned environment.

Do - advise the patient how to handle the pack - especially how to break the foil from the reverse or using an Omnicell Pilbob in case their fingers/hands are infirm. A separate instruction sheet is available for patients.

Do - only remove half the backing paper to reveal the blister cavities and immediately insert the blisters. Only when you are ready to seal the card should you remove the other half of the backing paper.



DON'Ts

Don't - apply pressure by hand - always use the roller.

Don't - fold or crease the card except where intended along the score lines.

Don't - fold against the fold lines.

Don't - allow the pack to become damp or wet before, during or after assembly.

Don't - leave the glue on the back of the card exposed (by removing the backing paper) it should either have the blister adhered to it or the card should be fully sealed.

If medication changes are needed, sealed packs can be corrected by using pill pack repair patches - also available from Omnicell.

1: Carefully pierce and remove the paper from the rear of the blister that needs to be changed.

2: Amend the medication and check as appropriate.

3: Carefully reseal the blister using a pill pack repair patch. The adhesive edge around the patch should be firmly pressed over the rear edge of the blister cavity to form a new seal.

Pill pack patches can be purchased from Omnicell and are available in packs of 200 patches.

TOP TIP:
For the perfect seal always place the filling template on a flat, even surface and remember to apply pressure using a roller!



You will need

SureMed by Omnicell weekly pill pack



Blisters



Filling template



Roller



Storage advice

We want to make sure our products perform perfectly every time. We always recommend that our pill packs should be stored in their original packaging and in a dry environment. Storage temperatures should not exceed the range - 5°C to 37.5°C, be stored in direct sunlight, and should be kept away from sources of heat, and ignition. If stored properly, our products are expected to provide a quality performance for 24 months from date of production. Production dates are clearly printed on each box. It's important to rotate stock and check storage conditions meet this guidance. Please allow packs to reach room temperature before assembly (approx. 21°C).

Inspired by Care: We see ways to save the planet everyday

Omnicell is a socially and environmentally friendly company. The card in this product is sourced from legal and responsible sources. The plastic blister is made from recycled materials. This hard filling template is fully recyclable. The recycled plastic used in our products is fully food grade with BRC accreditation for all suppliers of our plastic products. To find out more visit www.omnicell.co.uk

Contact us today to find out more

0161 413 5333
ordersuk@omnicell.com
www.omnicell.co.uk

