

# How to use the SureMed by Omnicell weekly pill packs

Each weekly pill pack will help you take the right pills and right dose at the right time.


There are a number of ways your medication can be removed from your weekly pill pack.

Firstly open your weekly pill pack and locate the correct day and time.



## Puncture method

- 1: Puncture the seal using a finger or fingernail.
- 2: Tip your weekly pill pack over your hand or other suitable container.
- 3: The medication that has been removed can now be taken.

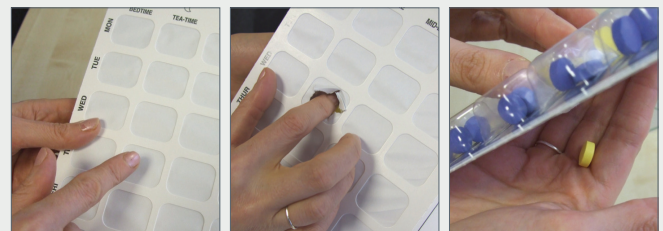

## Using a Pilbob® method

- 1: Hold the pack upright and puncture the seal using the sharp tip of the pilbob.
- 2: Move the pilbob from corner to corner following the edge of the blister.
- 3: All the medication will fall into the pilbob and can now be taken.

